



## Intersurgical i-flo™ High flow nasal cannula



**Critical Care** ▪ i-flo - High flow nasal cannula



## Intersurgical i-flo™ - High flow nasal cannula

The i-flo cannula is Intersurgical's single-use patient interface designed for the delivery of High Flow Oxygen Therapy (HFOT). The i-flo cannula is available in three adult sizes and is for use with patients in a hospital environment.


HFOT has become a key mode of respiratory support in patients with hypoxemic respiratory failure and is considered to offer a number of benefits versus more conventional oxygen therapies. These benefits have been shown to include:

- Improved oxygenation
- Reduced work of breathing
- Improved secretion clearance
- Improved patient tolerance
- Avoidance of intubation<sup>[1]</sup>

The i-flo cannula complements Intersurgical's extensive range of Critical Care breathing systems and interfaces. The unobtrusive design has been developed with a focus on performance and patient comfort.

The i-flo includes a number of unique features and benefits to help ensure the optimal delivery of high flow oxygen therapy with a flow rate of up to 60 L/min.


### Key features

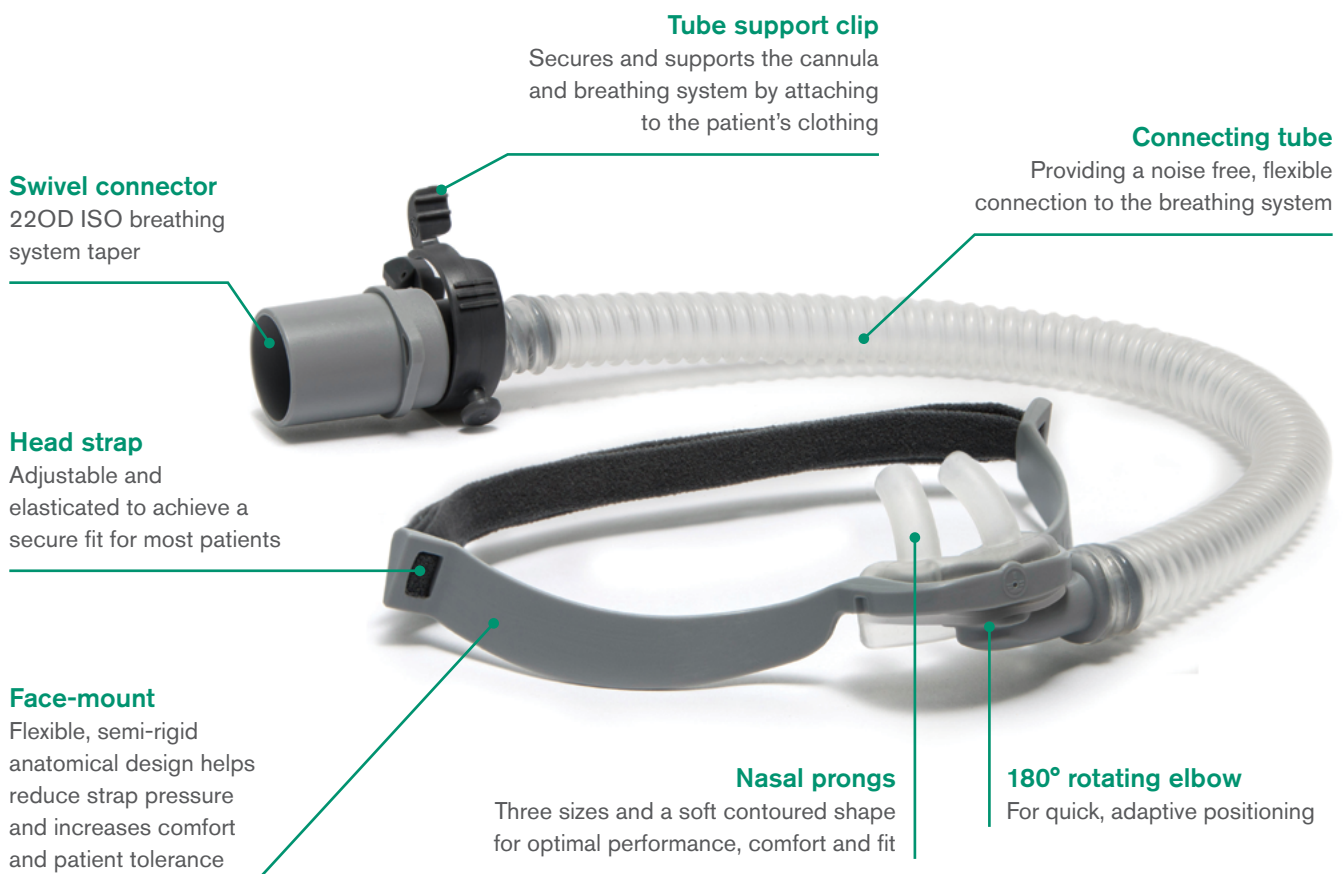

[1] Masaji Nishimura. High-Flow Nasal Cannula Oxygen Therapy in Adults: Physiological Benefits, Indication, Clinical Benefits, and Adverse Effects. Resp.Care 2016 Apr;61(4):529-41



## Adaptive and secure

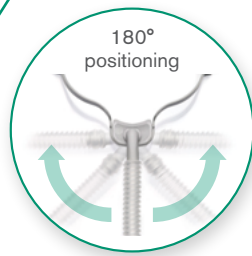

The i-flo's unique, 180° swivel elbow enables easy and convenient positioning of bedside equipment.

The i-flo clip and lanyard tube management system securely supports the breathing system tubing. This allows the patient to move freely without concerns of cannula dislocation or discomfort.

## Comfortable and supportive



An anatomical face-mount design directs the angle of the head strap above the patient's ears, helping to reduce trauma to the top of the ears.

The flexible, semi-rigid face-mount also ensures that strap pressure is distributed evenly over the patient's cheeks. This increases cannula stability and support helping to prevent pressure points from developing.






## Intersurgical i-flo™ cannula



Code	Description	Qty.
7300000	i-flo high flow nasal cannula, small adult	7
7301000	i-flo high flow nasal cannula, medium adult	7
7302000	i-flo high flow nasal cannula, large adult	7

- [Make an inquiry](#)
- [Find out more](#)
- [Watch the video](#)

## Single limb breathing systems

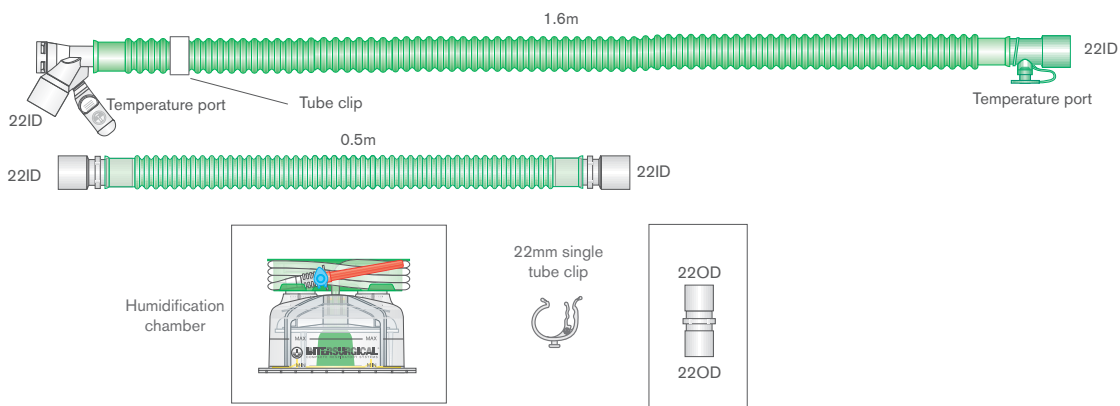

illustrated

Code	Description	Qty.
2026035	22mm Flextube single limb heated wire with limb, ≥ 1.6m length	20
2026036	22mm Flextube single limb heated system with auto-fill chamber and limb, ≥ 1.6m length	7

[Make an inquiry](#)