



# Intersurgical TrachSeal™

Closed suction systems



**Critical Care** ▪ TrachSeal

## TrachSeal™ Closed suction systems

Every time any part of a breathing system is opened to the atmosphere a number of problems for both the patient and nursing staff can occur:

- There is an immediate loss of PEEP to the patient and a potential for compromising patient ventilation
- Oxygen desaturation may have an impact on the patient's overall condition
- Bacteria and other pathogens may find their way into the lungs with the introduction of an open catheter
- A coughing reflex by the patient may put the Healthcare provider at risk for cross-infection

Since the introduction of closed suction systems in 1987, Healthcare providers have been offered a solution to keep the breathing system, and consequently PEEP, intact resulting in patient ventilation uncompromised.

Closed suction enables the clinician to clear the lungs of secretions while maintaining ventilation and minimizing contamination with the least possible disruption to the patient.

### Prevention of VAP

Ventilator associated pneumonia (VAP) is a hospital acquired infection, which can be efficiently fought when applying certain strategies. It has a significant impact on patient's health, length of stay and hospital cost.

Closed suction greatly reduces the potential for contamination from the bacteria which cause VAP. Maintaining a closed breathing system is part of the ventilator care bundle, designed to reduce the incidence of VAP in the hospital.

Another important factor in infection control is the cough reflex of the patient. By using a closed suction system the care provider reduces the risk of cross infection by aerosolized pathogens.

### Intersurgical TrachSeal closed suction systems

TrachSeal features an ultra clear, new softer sleeve, allowing the clinician a greater feel when inserting the catheter with optimum visibility of blockage assessments.

Our suction control valve is responsive and allows for single handed control of the applied suction force. The positive locking mechanism is designed to prevent suction being accidentally applied.



### Adult TrachSeal closed suction systems


Trachseal closed suction systems are available for 24 hour or 72 hour usage depending on patient requirements. There are two length options to allow attachment to either endotracheal or tracheostomy tubes. The endotracheal products feature a catheter length of 54cm and the tracheostomy tube variants have a catheter length of 30.5cm.

All Intersurgical TrachSeal systems are supplied with:

1. Day stickers which easily identify change requirements
2. 15ID –22ID catheter mount for flexible connection to the patient
3. A disconnection wedge for removal of the closed suction system from the endotracheal and tracheostomy tubes with minimal discomfort to the patient



[▶ Watch the video](#)

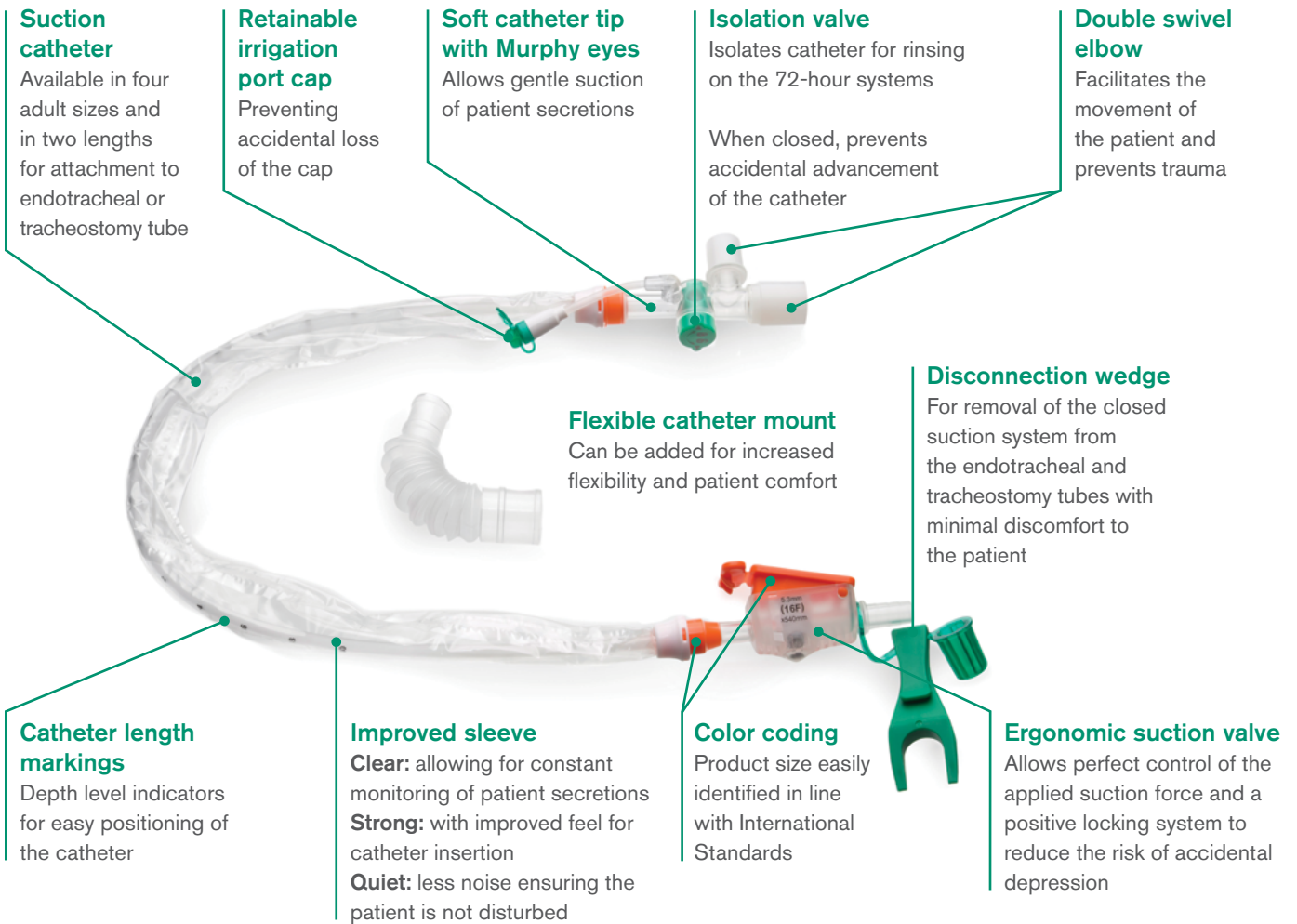

**24 hour TrachSeal™**

Code	Description	Size	Qty.
● 3240002	TrachSeal adult <b>endotracheal</b> closed suction system, 24 hour	16Fr	15
● 3240001	TrachSeal adult <b>endotracheal</b> closed suction system, 24 hour	14Fr	15
○ 3240000	TrachSeal adult <b>endotracheal</b> closed suction system, 24 hour	12Fr	15
● 3240003	TrachSeal adult <b>endotracheal</b> closed suction system, 24 hour	10Fr	15
● 3240007	TrachSeal adult <b>tracheostomy</b> closed suction system, 24 hour	16Fr	15
● 3240006	TrachSeal adult <b>tracheostomy</b> closed suction system, 24 hour	14Fr	15
○ 3240005	TrachSeal adult <b>tracheostomy</b> closed suction system, 24 hour	12Fr	15
● 3240004	TrachSeal adult <b>tracheostomy</b> closed suction system, 24 hour	10Fr	15

**72 hour TrachSeal**

Code	Description	Size	Qty.
● 3720002	TrachSeal adult <b>endotracheal</b> closed suction system, 72 hour	16Fr	15
● 3720001	TrachSeal adult <b>endotracheal</b> closed suction system, 72 hour	14Fr	15
○ 3720000	TrachSeal adult <b>endotracheal</b> closed suction system, 72 hour	12Fr	15
● 3720003	TrachSeal adult <b>endotracheal</b> closed suction system, 72 hour	10Fr	15
● 3720007	TrachSeal adult <b>tracheostomy</b> closed suction system, 72 hour	16Fr	15
● 3720006	TrachSeal adult <b>tracheostomy</b> closed suction system, 72 hour	14Fr	15
○ 3720005	TrachSeal adult <b>tracheostomy</b> closed suction system, 72 hour	12Fr	15
● 3720004	TrachSeal adult <b>tracheostomy</b> closed suction system, 72 hour	10Fr	15



### 24 hour TrachSeal™ with saline ampules

Code	Description	Size	Qty.
● 3241002	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 24hour	16Fr	15
● 3241001	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 24hour	14Fr	15
○ 3241000	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 24hour	12Fr	15
● 3241003	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 24hour	10Fr	15
● 3241007	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 24hour	16Fr	15
● 3241006	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 24hour	14Fr	15
○ 3241005	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 24hour	12Fr	15
● 3241004	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 24hour	10Fr	15

### 72 hour TrachSeal with saline ampules

Code	Description	Size	Qty.
● 3721002	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 72hour	16Fr	15
● 3721001	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 72hour	14Fr	15
○ 3721000	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 72hour	12Fr	15
● 3721003	TrachSeal adult <b>endotracheal</b> closed suction system with saline ampules, 72hour	10Fr	15
● 3721007	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 72hour	16Fr	15
● 3721006	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 72hour	14Fr	15
○ 3721005	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 72hour	12Fr	15
● 3721004	TrachSeal adult <b>tracheostomy</b> closed suction system with saline ampules, 72hour	10Fr	15

Make an inquiry

### Accessory

Code	Description	Qty.
3022000	Suction line splitter for use with closed suction and oral care line	100

Make an inquiry