



# VT Select Bag-Valve-Mask (BVM) resuscitator



Anesthesia • Resuscitation and Emergency Care



Quality, innovation and trust

Interact with us



[www.intersurgicalinc.com](http://www.intersurgicalinc.com)

## VT Select BVM resuscitator

As demonstrated by previous studies<sup>1</sup>, manual resuscitation can be highly variable from provider to provider. The VT Select helps to minimize the variability in treatment by using a timed refill valve at the back of the bag.

It is recommended by the American Heart Association<sup>2</sup> to deliver 10 breaths per minute (BPM) of 500-600ml in adult resuscitation. When activated, the valve restricts the refill of the bag to approximately 4 seconds, providing approximately 10 BPM to the patient, helping to eliminate the possibility of hyperventilation.

When the valve is activated, a one handed squeeze of the bag will deliver approximately 500-600ml to the patient. Alternatively, the valve can be switched to standard, removing the delayed refill and the BVM can be used as normal.

The VT Select is smaller than most adult bags at only 1200ml, which also helps to reduce the risk of hyperinflation or barotrauma.

### Specifications

Body mass range	>40kg
Delivered Volume Range	(1 hand) 530ml / (2 hand) 690ml

### Optional accessories

- Adjustable PEEP valve
- Dial manometer
- BVM mask strap



### Clear anatomical cushion mask

For an effective seal and maximizes patient visibility



BVM resuscitators are single-use and latex free.




### Reservoir bag and Oxygen tubing

### Valve

**Activated position** - the valve is restricted to produce a controlled tidal volume and timed refill.  
**Standard position** - the valve has no restriction and is acting as a standard BVM

### Smaller volume bag

The 1200ml bag has raised finger ridges and integrated low profile strap

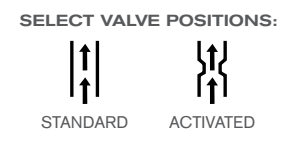

Code	Description	Box Qty
7159094	VT Select BVM Resuscitator, Adult, 1.2L Bag, Size 5 Mask	8
7159093	VT Select BVM Resuscitator, Adult, 1.2L Bag, Dial Manometer, Size 5 Mask	8
7159092	VT Select BVM Resuscitator, Adult, 1.2L Bag, Adjustable PEEP Valve, Size 5 Mask	8
7159091	VT Select BVM Resuscitator, Adult, 1.2L Bag, Adjustable PEEP Valve, Dial Manometer, Size 5 Mask	8
7159090	VT Select BVM Resuscitator, Adult, 1.2L Bag, Adjustable PEEP Valve, Dial Manometer, Filter, Size 5 Mask	8

### Reference:

1. American Medical Association, Benjamin S. Abella, MD, MPhil Et Al, 2005. Western Journal of Emergency Medicine, Benjamin Dafilou, BA Et Al, 2020.
2. American Heart Association, 2020. www.ahajournals.org/doi/epub/10.1161/CIR.0000000000000916

**Please think before you print** Save energy and paper. If you must print this information sheet please print it double sided.

IS5.41 US • Issue 2 01.25



Intersurgical Inc, 2055 Executive Dr. Indianapolis, IN 46241, USA

T: 800-828-9633 F: 315-451-3696 support@intersurgicalinc.com www.intersurgicalinc.com