



i-gel[®] O₂ Resus Pack

Three things in one pack, one less thing to think about



Airway Management ▪ Airway Devices



Quality, innovation and choice

Interact with us



www.intersurgicalinc.com



In emergency medicine, you need equipment that's easy, rapid and reliable to use. The i-gel® O₂ Resus Pack contains everything you need to prepare, insert and secure the i-gel® O₂ quickly and efficiently: an i-gel® O₂ supraglottic airway, a sachet of lubricant, airway support strap and a suction tube.


The i-gel® O₂ incorporates a supplementary oxygen port and an integral hook ring for securing of the airway support strap.

The i-gel® O₂ has been designed to facilitate ventilation as part of standard resuscitation protocols, such as those designated by the European Resuscitation Council (ERC) and the American Heart Association (AHA). However, the i-gel® O₂ incorporates a supplementary oxygen port, so can also be used for the delivery of passive oxygenation, or Passive Airway Management (PAM™), as part of an appropriate CardioCerebral Resuscitation (CCR) protocol.



Airway innovation since 2007

i-gel® is the innovative second generation supraglottic airway device from Intersurgical launched in 2007. The first major development since the laryngeal mask airway, the i-gel® changed the face of airway management and is now widely used in anesthesia and resuscitation around the world.

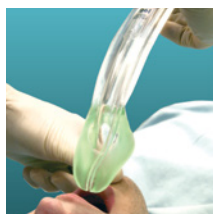

The non-inflatable cuff

The i-gel® gets its name from the innovative soft, gel-like material from which it is made. It is the innovative application of this material that has enabled the development of a unique non-inflatable cuff. This means there is no need for cuff deflation prior to insertion and no cuff inflation after placement to secure a seal, shortening and simplifying the preparation and insertion procedure.

Insertion

The i-gel® O₂ is incredibly easy to use. Insertion is rapid and can normally be achieved in less than 5 seconds. The simplicity of i-gel® O₂ means the insertion technique is instinctive, reliable and does not require or benefit from the insertion of fingers or thumbs into the patient's mouth.

[▶ Watch the video](#)

Fixation

The i-gel® O₂ Resus Pack includes a specially designed airway support strap for securing the i-gel® O₂ in position. Fixation is easily achieved by sliding the strap under the patient's neck and securing it in place to the hook ring on the collar of the i-gel® O₂. This makes it ideal for use where adhesive tape is unsuitable, such as on patients with facial hair.

[▶ Watch the video](#)



i-gel® O₂ Resus Pack

15mm connector

Reliable connection to any standard catheter mount or connection

Proximal end of gastric channel

Color coded hook ring

To secure the i-gel® O₂ in position with the airway support strap

Clearly displayed product information

For quick easy reference. Includes confirmation of size and weight guidance

Position guide

Supplementary oxygen port

For the administration of passive oxygenation as a component of cardiocerebral resuscitation (CCR)

[▶ Watch the video](#)

Gastric channel

Enhances patient safety by providing a mechanism for the management of regurgitant fluid

Integral bite block

Reduces the possibility of airway channel occlusion

Buccal cavity stabilizer

Aids insertion and eliminates the potential for rotation

Epiglottic rest

Reduces the possibility of epiglottis 'down folding' and airway obstruction

Non-inflatable cuff

Eliminates the need for cuff inflation after insertion, allowing easy and rapid insertion

Distal end of gastric channel



The i-gel® O₂ Resus Pack is available in 3 adult sizes and includes:

- i-gel® O₂ supraglottic airway
- Sachet of lubricant – for quick and easy lubrication of the i-gel® O₂ prior to insertion
- Airway support strap – to secure the i-gel® O₂ in position




[▶ Watch the video](#)



Ordering information

The i-gel® O₂ Resus Pack contains an i-gel® O₂, a sachet of lubricant and airway support strap, all packaged in a specially designed clear, sterile, rigid pack. This unique design ensures protection of the contents, both in transit and in storage. The i-gel® O₂ has a color coded hook ring to allow quick and easy identification of size. The i-gel® O₂ Resus Pack is available in three adult sizes.



Code	Description	Color	Weight	Box Qty.
8705030	i-gel® O ₂ Resus Pack, large adult – includes a size 5 i-gel® O ₂ with orange hook ring, sachet of lubricant and airway support strap	Orange	90+kg (200+lbs)	6 
8704030	i-gel® O ₂ Resus Pack, medium adult – includes a size 4 i-gel® O ₂ with green hook ring, sachet of lubricant and airway support strap	Green	50–90kg (110–200lbs)	6 
8703030	i-gel® O ₂ Resus Pack, small adult – includes a size 3 i-gel® O ₂ with yellow hook ring, sachet of lubricant and airway support strap	Yellow	30–60kg (65–130lbs)	6 

[Make an inquiry](#)

[Find out more](#)

 Sterile

Please think before you print Save energy and paper. If you must print this information sheet please print it double sided.

U.S. Lit No. 100.5 / IS6.6 US • Issue 10 07.24



Intersurgical Inc, 2055 Executive Dr. Indianapolis, IN 46241, USA

T: 800-828-9633 F: 315-451-3696 support@intersurgicalinc.com www.intersurgicalinc.com

UK • Australia • Benelux • Brazil • Canada • China • Colombia • Czech Republic • Denmark • France • Germany • Ireland • Italy • Japan
Lithuania • Malaysia • Mexico • Norway • Philippines • Portugal • Russia • South Africa • Spain • Sweden • Taiwan • Turkey • USA